cutape ®

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Video manual

**[www.cutape.es](http://www.cutape.es)**

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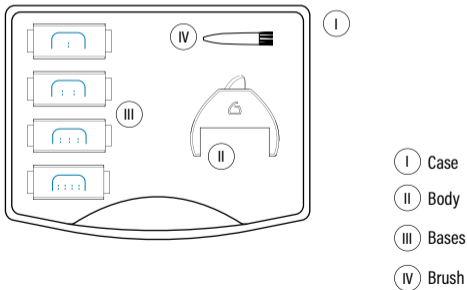
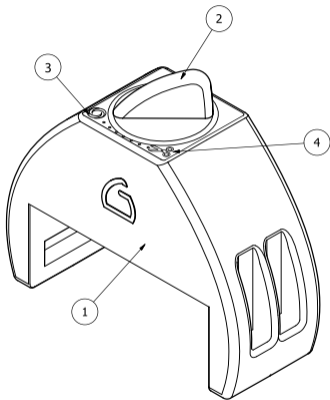
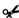
## Before you start

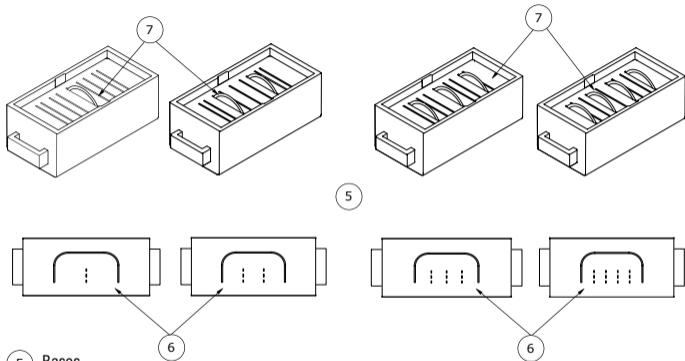
This product has been designed to make quick and equal cuts, in the several options of use of the neuro-muscular tape. With the aim of simplifying the cut, improve the runtime and the symmetry of the results.

The product contains:

- ① Body
- ② Switch
- ③ Resting position 
- ④ Cutting position 

5 Bases

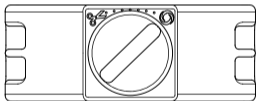
6 Signs

7 Blades

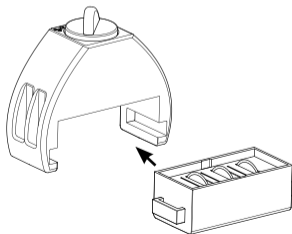
## Operating Instructions

- 1 Get the body (1) and place the switch (2) in the resting position  (3)

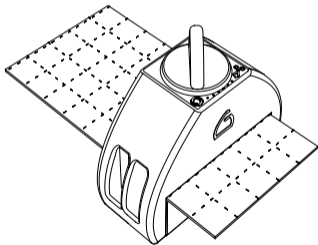
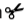
- 2 Choose a base (5) depending on the desired result; taking as reference the signs (6)

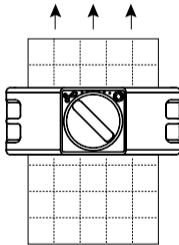
Insert the base in the body through the slits

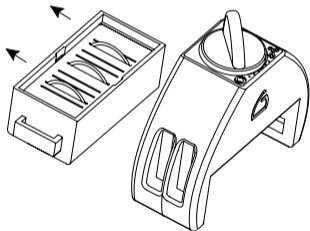
- 3 Enter the tape length to be cut through the groove with the colour side toward the blades (7)

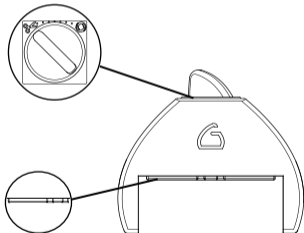
- 4 Turn the switch until the cutting position  (4) and pull the tape perpendicular.

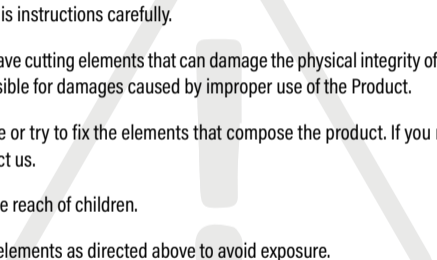
- 5 Turn the switch back on the resting position  and remove the base

- 6 We recommend rotating the switch to cutting position  when not to be used and always without a base in the body.

## Safety Precautions

- 
- ✓ Please, read this instructions carefully.
  - ✓ The product have cutting elements that can damage the physical integrity of the user. Cutape is not responsible for damages caused by improper use of the Product.
  - ✓ Do not remove or try to fix the elements that compose the product. If you need to repair it, please, contact us.
  - ✓ Keep out of the reach of children.
  - ✓ Store cutting elements as directed above to avoid exposure.

---

## Care and Maintenance

- ▶ It is recommended to clean frequently the bandage remnants from the cut on both the bases and the inside of the body.

DO NOT use any liquid that may damage the integrity of the components.

- ▶ Turn the switch to the cutting position whenever there is no inserted base in order to avoid prolonged internal stresses.
- ▶ Without disassembling the switch, lubricate the inner area of the switch with oils suitable for polymers (silicone, teflon, PTFE...), through the concavity between the switch and the body. Apply with the spray, wait for it to filter and remove the remaining liquid.

DO NOT remove the elements of the product. DO NOT use lithium-based grease or not suitable for polymers

- ▶ Always store the bases in the holes of the foam by placing the blades to the bottom of the foam case.
- ▶ Avoid long exposure to the sun and other external agents such as salty and humid environments.

---

## Legal Disclaimer

The information contained in this manual is provided for information only and may change at any time without notice, and should not be considered as an obligation for Cutape responsables. They take no responsibility for any mistakes or inaccuracies that may appear in this manual.

The products and trade names listed in this manual are trademarks and intellectual property titles.

## Recycling

The Cutape team is aware of the care of the environment. For it, we offer the possibility to send the product after the end of its useful life for the appropriate classification and recycling. Visit the website: [\*\*www.cutape.es/reciclado\*\*](http://www.cutape.es/reciclado) for more information. If you prefer to recycle the product by yourself, we recommend you, check the regulations of your region and take into the nearest recycling centre.

## Warranty

Cutape guarantees the end user or the buyer of this product that the product does not present material or manufacturing defects, in accordance with the conditions set out in the warranty, during the period of twenty four months from the date of purchase of the product in its original packaging. This warranty is regulated by the European Union legal warranty law.

For full warranty statement, visit [\*\*\*www.cutape.es/terms-compra-garantia/\*\*\*](http://www.cutape.es/terms-compra-garantia/)

## Customer Service

You can contact our Customer Care Service through our web [\*\*\*www.cutape.es/contacto\*\*\*](http://www.cutape.es/contacto) or, by e-mail to [\*\*\*info@cutape.es\*\*\*](mailto:info@cutape.es) or social networks

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*Marca registrada © 2019 todos los derechos reservados.